



2023.03.22 Ver.3

Patient Transfer System



Instruction for use

EZ-100P Patient Transfer Slide
















Important:

Read these instructions carefully before using the EZ-100P Patient Transfer Slide.

Manufacturer

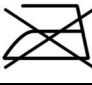EZ-Care Solution, Co., Ltd.
12F, No. 171, Sec. 4, Nanjing E. Rd.,
Songshan Dist., Taipei City 105, Taiwan
TEL: +886-2-2659-2277; FAX:+886-2-2798-8321;
E-mail: sales@ezcaresolution.com
www.ezcaresolution.com

All of the aids produced by EZ-Care Solution Co., Ltd fulfill the following norms for the medical products Class 1 (MDR (EU) 2017/145).

Explanation of symbols in Instruction for use and on labels			
	Model Number		Warning
	Lot No.		Maximum load
	Read the user manual		Manufacturer
	No sharps		European Representative
	Do not bleach		Medical device
	Do not iron		CE-marking
	Manual Cleaning		Unique Device Identification
	Keep out of the sun		

Product Label

EZ-Go®


Patient Transfer System


Patient Transfer Slide

REF EZ-100P


LOT 230301



 Do Not high
temp sterilize

 **MAX
LOAD** 200Kg

Material: Nylon
MADE IN TAIWAN

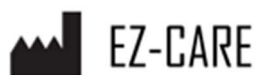

EZ-Care Solution Co., Ltd.
12F, No.171, Sec 4, Nanjing
E. Rd., Songshang Dist.,
Taipei Taiwan
www.ezcaresolution.com
sales@ezcaresolution.com

EC REP Obelis s. a.
Boulevard Général Wahis,
53 1030 Brussels, Belgium
Tel: +32 (2) 73 25 954
sales@obelis.net

MD 

UDI



471704851EZ-10034

Contents

1. Introduction	5
1.1 Intended Use	5
1.2 Intended Environment	5
1.3 Safety	5
1.4 Warranty	6
1.5 Product Structure	6
1.6 Product Specification	6
2. Operation	7
3. Cleaning Instruction	8
4. Maintenance	8

1. Introduction

1.1 Intended Use

EZ-100P is a lightweight, foldable, patient transfer device that helps move a patient from one flat surface to another, such as from a bed to an x-ray/MRI table, operating table, shower trolley, or ambulance stretcher.

1.2 Intended Environment

Acute Care, Long-term Care, Home Care

1.3 Safety



A failure to comply with the following safety points and further definitions in these instructions for use could lead to dangerous situations. Please read carefully.



Please consult with the patient's care team in advance to ensure the product can be used, and in combination with other devices, that is safe for the patient and the caregivers.



Always complete a patient risk assessment prior to all patient-handling activities.



It's important that caregivers receive manual handling training in advance.



There is a risk that the patient may slide off. Never leave the patient alone on the edge of the bed.



Never leave the product on the floor.



The maximum load of EZ-100P is 200 Kg.



The maximum safety gap is 15cm between the transfer surfaces; however, it is strongly recommended that two surfaces are leveled and as close as possible.




Ensure the transfer surfaces are securely locked.

1.4 Warranty

All EZ-GO products are supplied with a 1-year warranty, on construction and/or material faults unless otherwise stated. In the event of varying warranty terms, the warranty starts from the invoice date. The warranty does not cover failures and damage not caused by the product's defects such as improper cleaning.

1.5 Product Structure

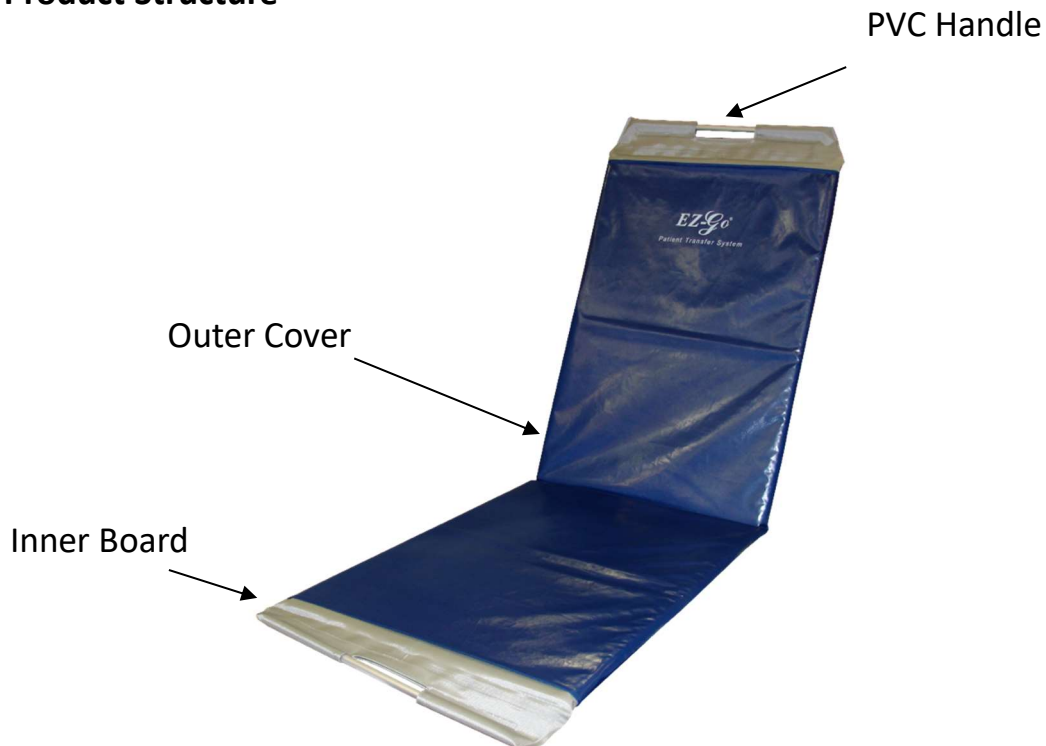

1.6 Product Specifications

Size: 180 x 56 cm ($\pm 3\%$) (unfolded)

88 x 56 cm ($\pm 3\%$) (folded)

Weight: 2.2 Kg ($\pm 3\%$)

Material:

Textile: Nylon

Handle: PVC

Inner Board: EVA


Lifetime: under normal storage and use condition, 3 years.

2. Operation

- Unfold EZ-100 and push it adjacent to the patient.
- Lean the patient to one side and roll EZ-100 underneath the patient.
- Lay the patient on the slide.



- Gently push the patient's shoulder and hip so EZ-100 can easily roll across the transfer surfaces. Ensure the patient is fully transferred to the other device.

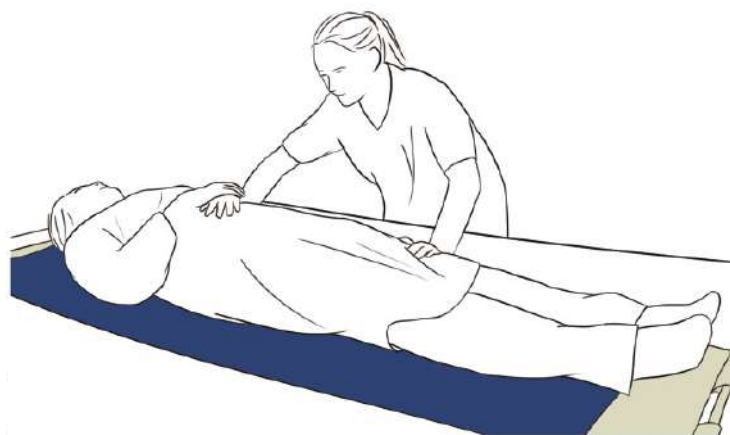


- Lean the patient to the other side and roll out EZ-100. Remove the slide.



3. Cleaning Instruction

- Hand clean with water and regular household cleaning agents.
- Use 70% ethanol to disinfect the product if necessary.
- Do not iron.
- Do not high-temperature sterilization.

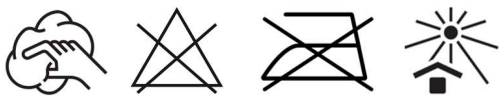

4. Maintenance

- Store at room temperature and keep out of the sun.
- EZ-100 must be checked periodically.
 - ✓ Visual Check: Ensure the materials, seams, and stitching are intact/faultless.