



2023.03.22 Ver.3

*Patient Transfer System*



## Instruction for use

EZ-221 Transfer Slide Sheet















## Important:

Read these instructions carefully before using the EZ-221 Transfer Slide Sheet.

## Manufacturer

EZ-Care Solution, Co., Ltd.  
12F, No. 171, Sec. 4, Nanjing E. Rd.,  
Songshan Dist., Taipei City 105, Taiwan  
TEL: +886-2-2659-2277; FAX:+886-2-2798-8321;  
E-mail: sales@ezcaresolution.com  
[www.ezcaresolution.com](http://www.ezcaresolution.com)

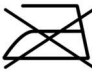All of the aids produced by EZ-Care Solution Co., Ltd fulfill the following norms for the medical products Class 1 (MDR (EU) 2017/145).

Explanation of symbols in Instruction for use and on labels			
	Model Number		Warning
	Lot No.		Hang dry
	Read the user manual		Manufacturer
	No sharps		European Representative
	Do not bleach		Medical device
	Do not iron		CE-marking
	Machine washable 30°C		Unique Device Identification

## Product Label

**EZ-Go**<sup>®</sup>


*Patient Transfer System*

**Transfer Slide Sheet**

**REF** EZ-221


**LOT** 230301



 Do Not high  
temp sterilize

Material: Nylon

MADE IN TAIWAN

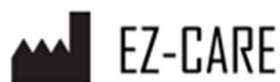

EZ-Care Solution Co., Ltd.  
12F, No.171, Sec 4, Nanjing  
E. Rd., Songshang Dist.,  
Taipei Taiwan  
[www.ezcaresolution.com](http://www.ezcaresolution.com)  
[sales@ezcaresolution.com](mailto:sales@ezcaresolution.com)

**EC REP** Obelis s. a.

Boulevard Général Wahis,  
53 1030 Brussels, Belgium  
Tel: +32 (2) 73 25 954  
[sales@obelis.net](mailto:sales@obelis.net)

**MD** 

**UDI**



## Contents

1. Introduction .....	5
1.1 Intended Use .....	5
1.2 Intended Environment .....	5
1.3 Safety .....	5
1.4 Warranty .....	6
1.5 Product Structure .....	6
1.6 Product Specification .....	7
2. Operation .....	7
2.1 Placement .....	7
2.2 Reposition in Bed .....	7
2.3 Lateral Transfer .....	8
2.4 Transfer between bed and wheelchair.....	9
2.5 Turn in bed .....	9
3. Cleaning Instruction .....	10
4. Maintenance .....	10

## 1. Introduction

### 1.1 Intended Use

EZ-221 is a lightweight, foldable, patient transfer slide sheet that helps move or reposition a patient in bed or transfer to another device. It can also be used with other devices to make the transfer smoother.

### 1.2 Intended Environment

Acute Care, Long-term Care, Home Care

### 1.3 Safety



A failure to comply with the following safety points and further definitions in these instructions for use could lead to dangerous situations. Please read carefully.



Please consult with the patient's care team in advance to ensure the product can be used, and in combination with other devices, that is safe for the patient and the caregivers.



Always complete a patient risk assessment prior to all patient-handling activities.



It's important that caregivers receive manual handling training in advance.



There is a risk that the patient may slide off. Never leave the patient alone on the edge of the bed.



Never leave the product on the floor.



The device is made of heavy-duty fabric with strong abrasion resistance, tensile strength, and tear strength for easy patient transfer.



When transferring to another device (such as from bed to trolley), ensure there is no gap between two surfaces and two surfaces are leveled.




Ensure the transfer surfaces are securely locked.

### 1.4 Warranty

All EZ-GO products are supplied with a 1-year warranty, on construction and/or material faults unless otherwise stated. In the event of varying warranty terms, the warranty starts from the invoice date. The warranty does not cover failures and damage not caused by the product's defects such as improper cleaning.

### 1.5 Product Structure

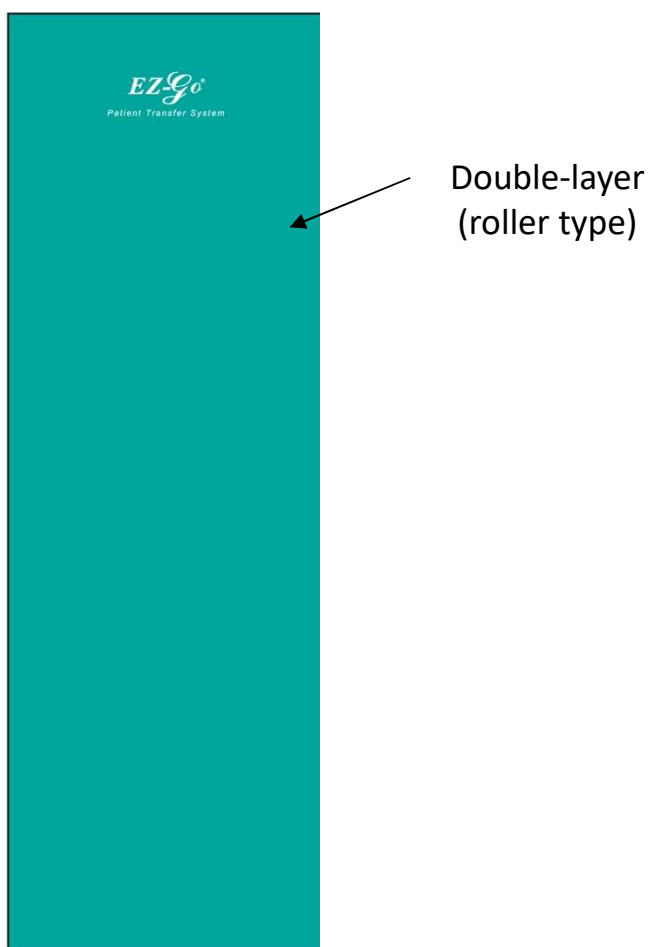

### 1.6 Product Specifications

Size: 170 x 75 cm ( $\pm 3\%$ ) (unfolded)

Weight : 170g ( $\pm 3\%$ )


Material: Nylon

Lifetime: under normal storage and use condition, 1 year.

## 2. Operation



### 2.1 Placement

- Unfold EZ-221 and lean the patient to one side. Place it along the patient's back  
Lean the patient to the other side and pull the slide out.

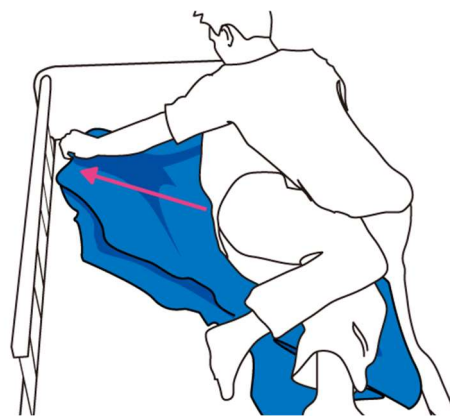


### 2.2 Reposition in bed

- Place EZ-221 under the patient and gently push/pull the patient to reposition in bed. Ensure the caregiver is trained for manual patient handling.

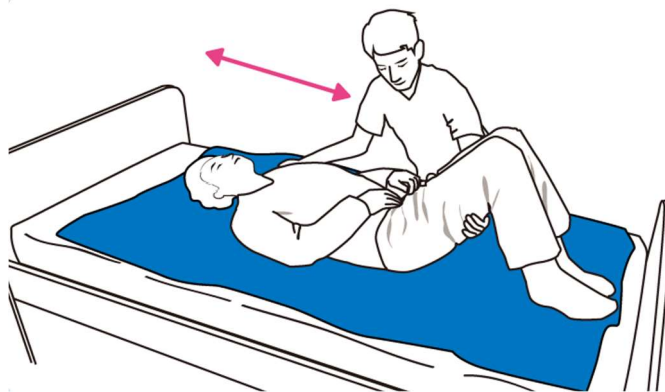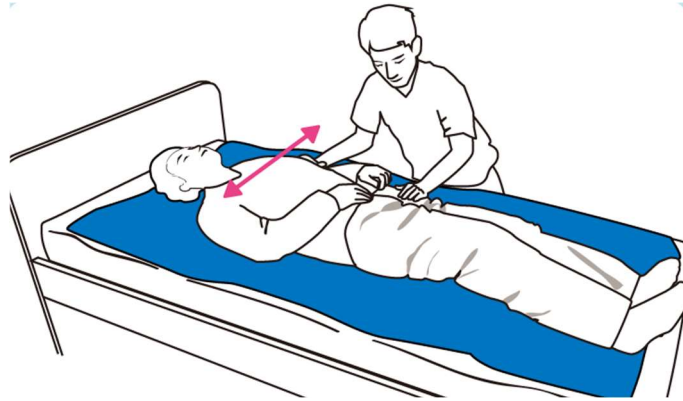

### 2.3 Lateral Transfer

- Place EZ-221 under the patient and gently push the patient to another device. A patient transfer board can be used in combination with EZ-221 for lateral transfer if needed.

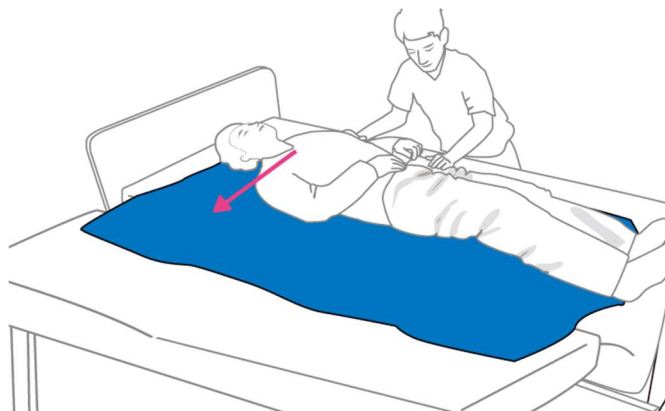


Ensure there is no gap between two surfaces and two surfaces are leveled.




## 2.4 Transfer between bed and wheelchair

- EZ-221 can be folded into smaller sizes. It can be used in combination with EZ-510 Butterfly Patient Transfer Board and the EZ-900 Patient Handling Belt series to make a transfer between bed and wheelchair easier and smoother.

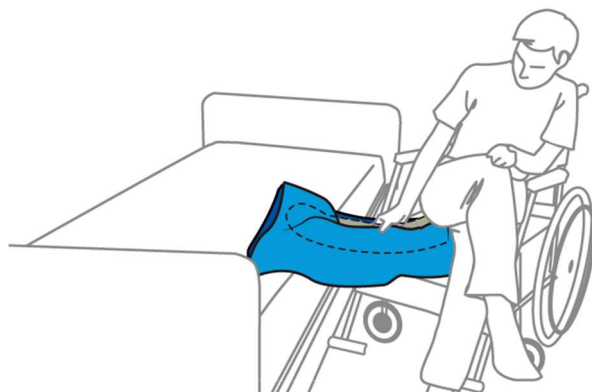


## 2.5 Turning in bed

- Place EZ-221 under the patient. Use both hands to grab the edge of the slide sheet near the patient's head and hip area, and pull to turn the patient.

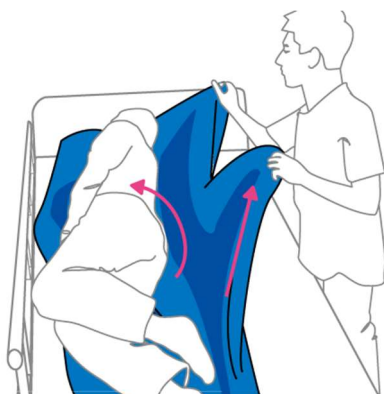

### 3. Cleaning Instruction

- Machine wash with temperature up to 30°C.
- Do not bleach.
- Do not iron.
- Do not high-temperature sterilization.

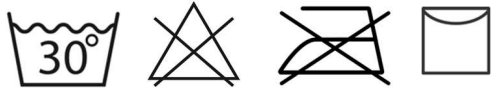

### Maintenance

- Store at room temperature.
- EZ-221 must be checked periodically.
  - ✓ Visual Check: Ensure the materials, seams, and stitching are intact/faultless.